

# cast protector (lower leg)

## User Instructions:

1. Check plaster casts for edges that may tear or puncture the cast protector.
2. Hold the plastic ring firmly, then pull over cast or bandage. Be sure the cast protector completely covers the cast or bandage and fits snugly around limb to form a secure seal.
3. BEFORE REMOVAL be sure to dry the diaphragm of the cast protector.
4. When removing be sure to grasp the plastic ring and NOT the vinyl (PVC) bag as this may cause tearing.

## CAUTIONS:

- This product is intended for single patient use and is not to be used as a flotation device.
- The plastic ring is not designed to have heavy weight applied to it, DO NOT lean on ring at any time. Do not walk or stand on cast protector.
- Do not use this product for any purpose other than as a cast protector. Follow user instructions at all times.
- The cast protector may become slippery, especially when wet, so use extreme caution when bathing or showering.
- Be sure the diaphragm is not too tight around your limb, this may impede circulation.

## Instrucciones para el usuario:

1. Revise que el yeso no tenga filos que puedan romper o perforar el protector para el yeso.
2. Sostenga el anillo plástico firmemente y cubra el yeso o la venda. Asegúrese que el protector para el yeso cubra completamente el yeso o la venda y que se ajuste muy bien alrededor de la extremidad formando un sello seguro.
3. ANTES DE QUITARSELO asegúrese de secar el diafragma del protector para el yeso.
4. Cuando se lo quite, asegúrese de agarrar el anillo plástico y no la bolsa de vinilo (PVC) ya que la podría romper.

## PRECAUCIONES:

- Este producto es para ser usado por un solo paciente y no es para ser usado como un artículo para flotar.
- El anillo de plástico no está diseñado para aplicarle peso. NO LO PRESIONE en ningún momento. No camine ni se pare sobre el protector para el yeso.
- Sólo utilice este producto como un protector para el yeso. Siga las instrucciones para el usuario en todo momento.
- El protector para el yeso puede ser resbaloso, especialmente cuando está mojado, así que sea extremadamente cauteloso cuando se esté bañando o duchando.
- Asegúrese que el diafragma no esté muy apretado alrededor de la extremidad ya que puede impedir la circulación.



**Carex Health Brands**

© 2008  
PO Box 2526  
Sioux Falls, SD 57101-2526

Phone: 800-328-2935

Fax: 888-616-4297

www.carex.com

Made in Taiwan/Hecho en Taiwan

41-P202-00\_02

**P202-00**