

User Guide – 9010 Adjustable Bath Bench



Weight Capacity: 300 lbs

Instructions Prior to Use

How to Adjust the Height

- 1) Depress the brass knob release that is visible in the series of holes for vertical height adjustment.
- 2) Lengthen or shorten the legs to the desired height. Make sure all 4 legs are set at the same height.
- 3) Make sure brass knob pops back out to maintain desired height

How to Use

- 1) Place the bath bench in the shower or bathtub.
- 2) Make sure the seat is stable and not wobbly
- 3) Use the slots in the seat as hand holds to assist in getting up and down from the bath bench.

Care and Maintenance

- 1) Check regularly to make sure the legs are securely set.
- 2) Use a mild soap and water to clean.

LIFETIME LIMITED WARRANTY

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product. NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent. This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear".

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

1-800-557-6682

www.novaortho-med.com